Opal® Seal EN Fluoride Releasing Primer and Sealant

Description and Indications for Use:

- Opal® Seal is a light cure primer that is used when bonding orthodontic appliances to etched enamel. Opal Seal is 38% filled with proprietary GI particles and nano fillers that release fluoride during orthodontic treatment. When Opal Seal has been applied to the tooth surface the use of a UV "black" light will make the sealant illuminate showing that the resin containing fluoride is present.
- Direct Bonding Procedure**
NOTE: Working one quadrant at a time is recommended.
1. Prepare enamel
 - a. Use oil-free, water-based pumice to clean enamel surface.
 - b. Rinse, dry and isolate the tooth to be etched.
 - c. Securely attach functional tip to 35% phosphoric acid etchant gel.
 - d. Verify etchant flow prior to applying intra-orally.
 - e. Apply etchant to enamel and leave in place 15 - 30 seconds.
 - f. Do not allow etch to contact soft tissue.
 - f. Rinse and dry thoroughly and re-isolate. Enamel will appear chalky white.

2. Apply Opal Seal
- a. Remove luer cap and securely attach brush tip.
- b. Verify flow prior to applying intra-orally by slightly twisting pink end and moistening the tip.
- NOTE: If too much is expressed, dab excess on a pad.
- c. Apply a thin layer of Opal Seal to the etched enamel giving moving quickly from tooth to tooth.
- d. Gently air dry for 2 seconds per tooth to thin. Sealant should appear shiny, not pooled on the tooth surface.
- OPTIONAL: Before placing bracket, tack-cure Opal Seal for 5 seconds per tooth to avoid "skating" upon placement.
3. Apply bonding adhesive to the mesh side of the bracket and cure on tooth.
4. Remove excess adhesive from around the bracket base.
5. Light cure as per adhesive/light manufacturers' instructions.

Opal Seal is designed to be re-applied periodically during orthodontic treatment. Shining a UV "black" light on the tooth will indicate the presence of the sealant.

- Re-Application Procedure**
1. With orthodontic wire removed, ensure enamel surface is free of plaque and debris.
 - NOTE: For optimal treatment: clean enamel surface with an oil-free, water-based pumice.
 2. Follow Steps 1b-1 applying the 35% phosphoric acid etch to the exposed enamel and up to the base of the bracket for 15 - 30 seconds.
 - NOTE: Phosphoric acid etchants will not cause de-bonding of any appliances.
 3. Follow steps 2a-d.
 4. Light cure 10 sec. per tooth (20 sec. for lights with output-<600mW/cm2).

Removal Procedure

1. At the conclusion of treatment the remaining sealant should be removed with a fluted bur or an abrasive polisher using caution not to overheat the tooth.
2. The use of a UV "black" light will illuminate the sealant for ease of removal.

- Precautions:**
1. For professional use only.
 2. Carefully read and understand all instructions, precautions and MSDS before applying.
 3. Verify flow prior to applying intra-orally. If resistance is met, replace tip and re-check. Use only recommended tips.
 4. Dispose of tips and empty syringes properly. Tips are single use only.

5. Rec-pac syringe with Luer Lock cap and disinfect with an intermediate level disinfectant between uses.
6. Use quality curing lights to cure product. Uncured resin is deficient in its physical properties.
7. Keep product out of heat and sunlight.
8. Not all bonding agents and adhesives are compatible. Test before using products outside the Ultradent or any specific manufacturer's product family.
9. If skin or soft tissue is exposed to product, thoroughly wash area. If allergic reaction, dermatitis or rash develops, consult a physician.

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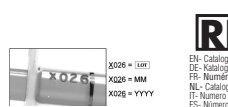
Our syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the month, and the third number is the last number of the year.

MPG Service GmbH, Gereonstraße 34-36, D-50670 Cologne, Germany

Opal® Seal
Fluoride Releasing Primer and Sealant

Manufactured by:
 Opal Orthodontics
 635 West 10200 South
 Jordan, Utah 84095 USA
 Toll Free: 1 (888) 863-5883
 Outside U.S. Call: +1(801) 553-4177

For product SDS and additional information about this product please go to www.opalorthodontics.com Made in USA

© Copyright 2011 Opal Orthodontics. All Rights Reserved. 50136.11 100195

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